


HEMOVAC[®]

**AUTOTRANSFUSION
SYSTEM**

*The Safe Solution In Wound
Drainage And Reinfusion*

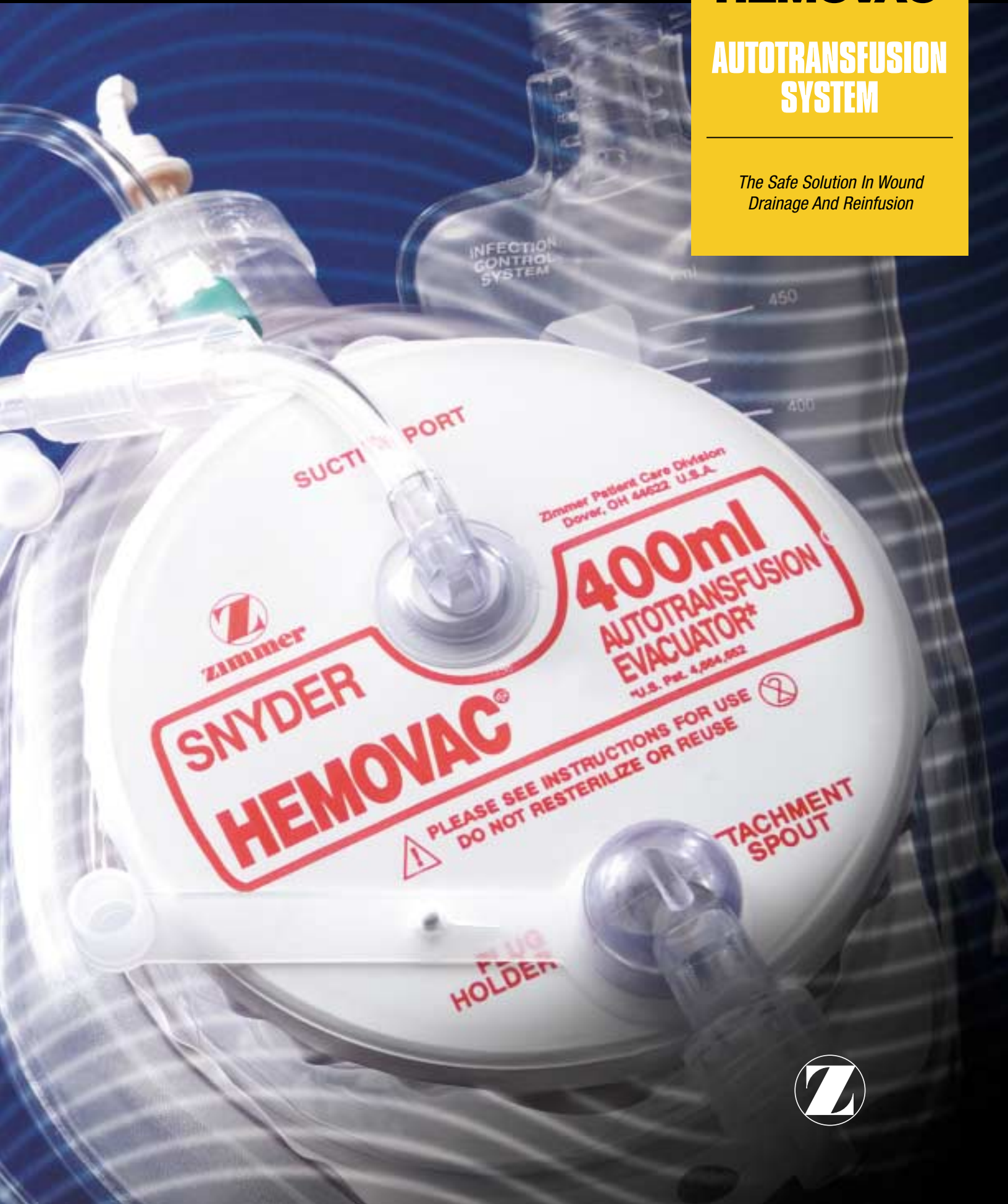

SUCTION PORT

Zimmer Patient Care Division
Dover, OH 44622 U.S.A.

400ml
AUTOTRANSFUSION
EVACUATOR*
U.S. Pat. 4,584,052



SNYDER
HEMOVAC[®]

PLEASE SEE INSTRUCTIONS FOR USE
DO NOT RESTERILIZE OR REUSE



PLUG
HOLDER

ATTACHMENT
SPOUT



A COMPLETE ORTHOPAEDIC SYSTEM FOR AUTOTRANSFUSION

The *Hemovac* Autotransfusion System (*H.A.S.*® Autotransfusion System) from Zimmer improves safety for the hospital staff and the patient. A complete system, the *H.A.S.* product provides



everything needed for post-operative salvage of the patient's own blood and for continued wound drainage after reinfusion: a collection unit, wound drainage tubing and an autotransfusion evacuator designed to minimize risk of spreading infection. As an infection control system, the *H.A.S.* product reduces the staff's exposure to blood and body fluids.

The *Hemovac* Autotransfusion System is a fully integrated, completely portable system that does not depend on external sources such as wall suction or batteries to operate. And because wall suction is not needed, the risk of hemolysis is greatly reduced. The system's low, gentle suction helps ensure the overall quality of blood to be reinfused.

EASY SET UP FOR EASY USE

The *Hemovac* Autotransfusion System has a minimal number of steps for assembly and operation. All connectors are identified for easy assembly. Just two quick attachments, compress the evacuator and drainage can begin. No batteries or other equipment are needed. Once drainage and reinfusion are completed, the entire system can be safely discarded.



Attachment Spout Cap

Autotransfusion Evacuator

Anti-Coagulant Port

Collection Bottle

Attachment Spout

Line Clamp

"Y" Connector

Exudate Collection Bag



Line Clamp

Collection Bottle Line

Evacuator Suction Port

Evacuator Tubing/Blood Line

Vent Tube

Filter Spike and Cap

MORE FEATURES FOR SAFETY AND CONVENIENCE

Superior design and thorough testing have made *Hemovac* the dependable name in wound drainage for more than three decades. This same expertise has been applied to reinfusion of the patient's own blood. Special attention is always given to the details designed to make the process safer and less complicated.

- A large 230 micron gross filter with an internal spring helps reduce clots and keeps blood flowing freely into the bottle.
- The system operates from any position during wound drainage which is especially convenient during patient transportation.
- The safety of a dual trocar drain assembly.
- Vents in the collection bag minimize retention of wound air, leaving good quality shed blood more room for reinfusion into the patient.
- A choice of three drain sizes, or none at all, is designed to accommodate any reinfusion and wound drainage need.

Whether in orthopaedic surgery or other specialty fields, turn to the reliable *Hemovac* product line for closed wound drainage innovation and reliability.

ORDERING INFORMATION

All *Hemovac* Autotransfusion System products are sold in box quantities of five (5) sterile kits.

Catalog No.	Description
2555-20-05	<i>Hemovac</i> Autotransfusion System with 1/8" (3mm), (10 French) PVC Drain
2555-30-05	<i>Hemovac</i> Autotransfusion System with 3/16" (5mm), (15 French) Soft PVC Drain
2555-40-05	<i>Hemovac</i> Autotransfusion System with 1/4" (6mm), (19 French) PVC Drain
2555-50-05	Replacement Bottle
2555-60-05	<i>Hemovac</i> Autotransfusion System without Drain

- Each complete kit contains wound drain with trocars, Y-connector, evacuator tubing, collection bottle bag and filter, plus an autotransfusion evacuator for infection control. Evacuator includes exudate collection bag. *H.A.S.* kit without drain contains evacuator tubing and a universal Y-connector along with the collection bottle, evacuator and exudate collection bag.



For more information, contact your Zimmer Representative.

