



**Load/Neutral
Drill Guide 2.7mm
00-2345-027-00**

**Assembly/Disassembly
Instructions**



Disassembly Instructions for Load/Neutral Drill Guide 2.7mm

1. Pull the Neutral Guide Assembly **2** firmly to disengage it from the Handle Assembly **1**.
2. Pull the Load Guide Assembly **3** firmly to disengage it from the Handle Assembly **1**.
3. End of disassembly.

Assembly Instructions for Load/Neutral Drill Guide 2.7mm

1. Push the Neutral Guide Assembly **2** firmly inside the Handle Assembly **1** until the spherical ball of the Neutral Guide Assembly **2** engages and free to rotate in the groove of the Handle Assembly **1**.
2. Push the Load Guide Assembly **3** firmly inside the Handle Assembly **1** until the spherical ball of the Load Guide Assembly **3** engages and free to rotate in groove of the Handle Assembly **1**.
3. End of assembly.



+H124970000504001/\$100610F10M

97-0000-504-00 1003-T29 Printed in USA ©2010 Zimmer, Inc.