



**Proximal Lateral
Tibial Left Jig
(3.5mm)**

00-2360-093-02

Assembly/Disassembly
Instructions

Proximal Lateral
Tibial Jig **1**



2 Jig Set Screw

Disassembly Instructions for Proximal Lateral Tibial Left Jig

1. Unthread the Jig Set Screw **2** out of the Proximal Lateral Tibial Jig **1**.
2. End of disassembly.

Assembly Instructions for Proximal Lateral Tibial Left Jig

1. Orient the flat surface of the Proximal Lateral Tibial Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Proximal Lateral Tibial Jig **1** fully until it moves freely inside the jig.
2. End of assembly.



+H124970000515001/\$100610F100

97-0000-515-00 1003-T17 Printed in USA ©2010 Zimmer, Inc.