




## Distal Medial Tibial Right Jig

00-2360-094-01

Assembly/Disassembly  
Instructions

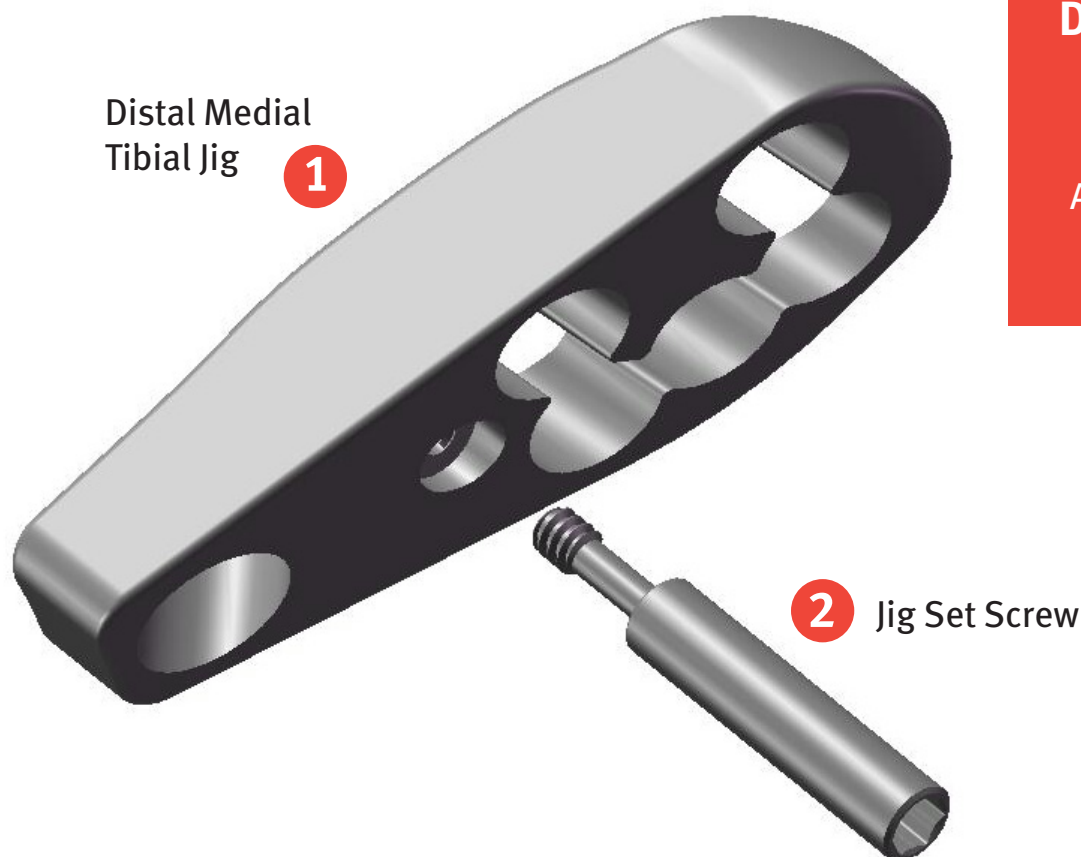

## Disassembly Instructions for Distal Medial Tibial Right Jig

1. Unthread the Jig Set Screw **2** out of the Distal Medial Tibial Jig **1**.
2. End of disassembly.

## Assembly Instructions for Distal Medial Tibial Right Jig

1. Orient the flat surface of the Distal Medial Tibial Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Distal Medial Tibial Jig **1** fully until it moves freely inside the jig.
2. End of assembly.



+H124970000516001/\$100610F10P

97-0000-516-00 1003-T18 Printed in USA ©2010 Zimmer, Inc.