



## Distal Lateral Tibial Right Jig

00-2360-095-01

Assembly/Disassembly  
Instructions



## Disassembly Instructions for Distal Lateral Tibial Right Jig

1. Unthread the Jig Set Screw **2** out of the Distal Lateral Tibial Jig **1**.
2. End of disassembly.

## Assembly Instructions for Distal Lateral Tibial Right Jig

1. Orient the flat surface of the Distal Lateral Tibial Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Distal Lateral Tibial Jig **1** fully until it moves freely inside the Jig.
2. End of assembly.



+H124970000517001/\$100610F10Q

97-0000-517-00 1003-T41 Printed in USA ©2010 Zimmer, Inc.