




## Drill Sleeve with Trocar (135mm)

00-4450-049-00

Assembly/Disassembly  
Instructions

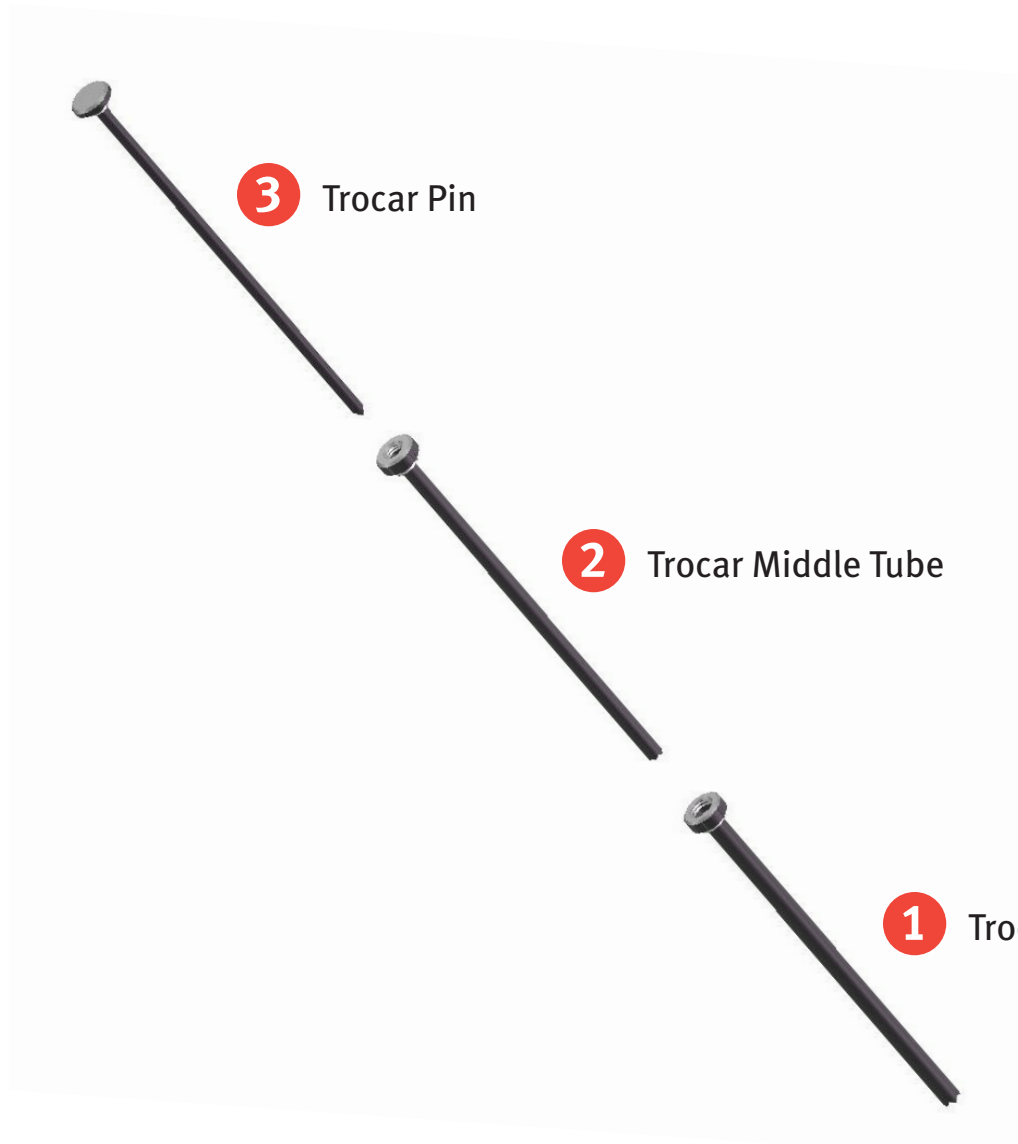

## Disassembly Instructions for Drill Sleeve with Trocar

1. Unthread the Trocar Pin **3** and pull out of Trocar Middle Tube **2**.
2. Unthread the Middle Tube **2** and pull out of Trocar Outer Tube **1**.
3. End of disassembly.

## Assembly Instructions for Drill Sleeve with Trocar

1. Insert the Trocar Middle Tube **2** inside the Trocar Outer Tube **1** and tighten completely.
2. Insert the Trocar Pin **3** inside the Trocar Middle Tube **2** and tighten completely.
2. End of assembly.



+H124970000523001/\$100610F10N

97-0000-523-00 1003-T35 Printed in USA ©2010 Zimmer, Inc.



zimmer | trauma