




ZNN™ Retrograde Femur Depth Gauge 00-2490-065-85

Assembly/Disassembly
Instructions

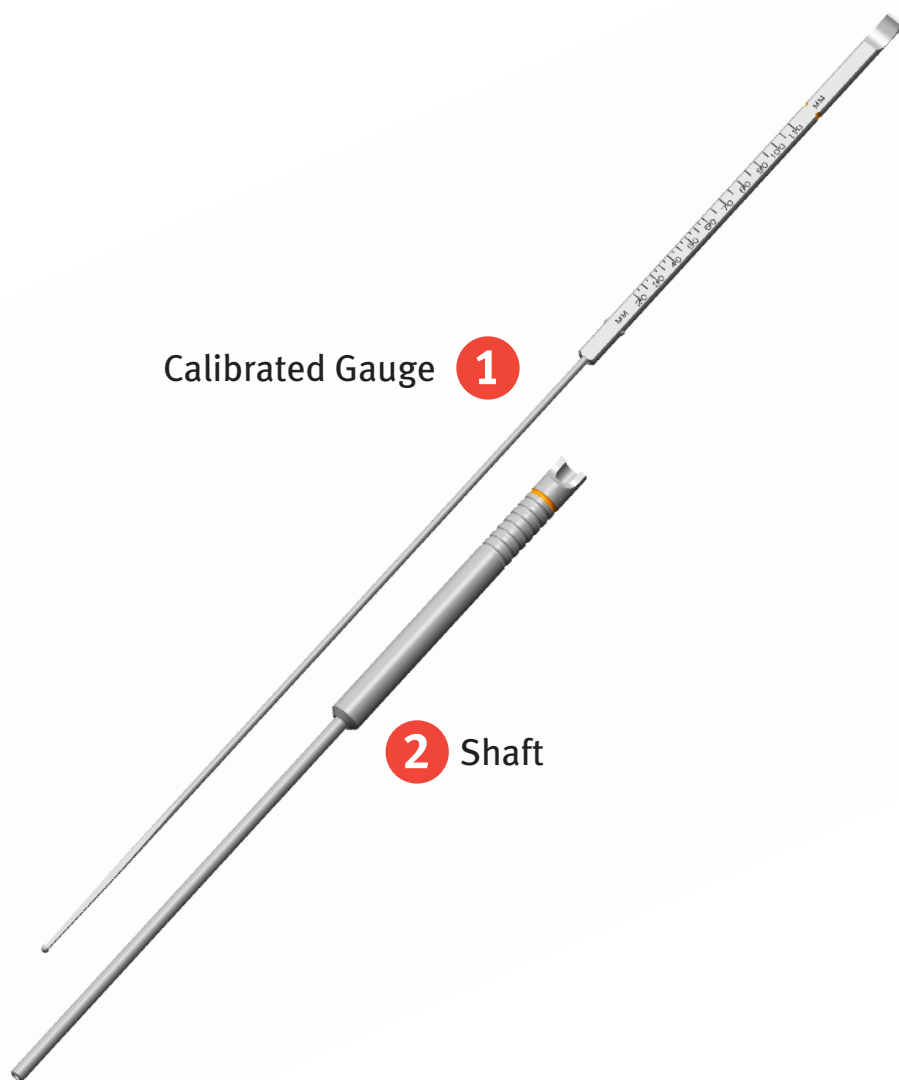

Calibrated Gauge **1**

2 Shaft

Assembly Instructions for Depth Gauge

1. Insert the Calibrated Gauge **1** into the Shaft **2**
2. End of assembly.

Disassembly Instructions for Depth Gauge

1. Remove the Calibrated Gauge **1** from the Shaft **2**
2. End of disassembly.



+H124970000525001/\$100929J10%

97-0000-525-00 1009-T12 Printed in USA ©2010 Zimmer, Inc.



zimmer | trauma